


# 3M™ Micropore™ Surgical Tapes

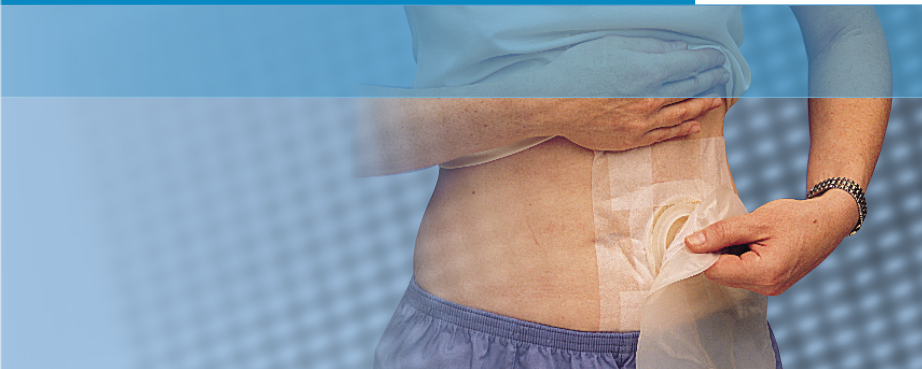

## Gentle, Reliable Securement



Now Available in  
Two Adhesion Levels

50 years ago, 3M introduced the industry's first hypoallergenic, breathable tape, a true breakthrough for patient safety and comfort. Today, 3M™ Micropore™ family of surgical tapes remain the #1 brand of surgical tapes globally. Micropore™ paper tapes:

- Offer gentle adhesion for at risk skin
- Are conformable and highly breathable
- Reliably secures dressings and devices to skin
- Are hypoallergenic and latex-free

#### Suggested applications:

- Securing small to medium dressings, especially on damp skin
- Securing light tubing
- Securing Ostomy pouches



## Ordering Information:

Catalog No.	NHRIC No.	Size - Standard	Size - Metric	Rolls/Box	Boxes/Case	HCPCS Code*
<b>3M™ Micropore™ Paper Tape Plus</b> 1532 has a higher initial adhesion than the 1530						
1532-1	8333-1532-01	1 in x 10 yd	2,5 cm x 9,1 m	12	10	A4450
1532-2	8333-1532-02	2 in x 10 yd	5 cm x 9,1 m	6	10	A4450
1532S-1	8333-1532-11	1 in x 1 ½ yd	2,5 cm x 1,37 m	100	5	A4450
<b>3M™ Micropore™ Surgical Tape</b>						
Standard:						
1530-0	8333-1530-00	½ in x 10 yd	1,25 cm x 9,1 m	24	10	A4450
1530-1	8333-1530-01	1 in x 10 yd	2,5 cm x 9,1 m	12	10	A4450
1530-2	8333-1530-02	2 in x 10 yd	5,0 cm x 9,1 m	6	10	A4450
1530-3	8333-1530-03	3 in x 10 yd	7,5 cm x 9,1 m	4	10	A4450
Single Use:						
1530S-1	8333-1530-11	1 in x 1 ½ yd	2,5 cm x 1,37 m	100	5	A4450
1530S-2	8333-1530-12	2 in x 1 ½ yd	5,0 cm x 1,37 m	50	5	A4450
Tan:						
1533-0	8333-1533-00	½ in x 10 yd	1,25 cm x 9,1 m	24	10	A4450
1533-1	8333-1533-01	1 in x 10 yd	2,5 cm x 9,1 m	12	10	A4450
1533-2	8333-1533-02	2 in x 10 yd	5,0 cm x 9,1 m	6	10	A4450
Dispenser Pack:						
1535-0	8333-1535-00	½ in x 10 yd	1,25 cm x 9,1 m	24	10	A4450
1535-1	8333-1535-01	1 in x 10 yd	2,5 cm x 9,1 m	12	10	A4450
1535-2	8333-1535-02	2 in x 10 yd	5,0 cm x 9,1 m	6	10	A4450
1535-3	8333-1535-03	3 in x 10 yd	7,5 cm x 9,1 m	4	10	A4450

\* HCPCS codes as referenced in the Local Coverage Determination Surgical Dressing Policy.

### Disclaimer:

HCPCS codes have been provided to assist you in the preparation of Medicare Part B claims. Please note, however, that the reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coverage and coding for 3M products. Insurers' reimbursement policies can vary and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level. Health care providers should exercise independent clinical judgment in choosing the codes which most accurately describe the products provided.

**Product Content:** Nonwoven rayon and hypoallergenic, pressure-sensitive, acrylate adhesive. 3M™ Micropore™ Tape is free of natural rubber latex.

**Storage/Shelf Life:** It is recommended that 3M™ Surgical Tapes be stored under normal conditions within the temperature range of 59–86°F (15–30°C). 3M™ Micropore™ Tape has a recommended shelf life of five (5) years under these conditions.

**Sterilization:** 3M™ Micropore™ Tape is sold non-sterile. The following methods were tested and proven reliable for sterilization and stable adhesion performance, with ethylene oxide residuals well within U.S. FDA guidelines. Procedure used: 1) Tape rolls kept intact up to the full 10-yard length, OR 2) Strips of tape arranged over a suitable release surface, e.g. glass, stainless steel, silicone-treated paper, 3) Tape sealed in appropriate sterilization pouch or wrapper, 4) EtO sterilized warm cycle 55°C (130–145°F), minimum 1 hour exposure time (50% RH - 600 mg/liter, or equivalent), 5) Warm cycle aeration (temperature not to exceed 55°C (130–145°F) for a minimum 20 hour cycle before use.

It is the responsibility of an individual health care organization to confirm sterility when product is sterilized in its facility.

**Application recommendations:** 1) Always apply to clean dry surface, 2) To maximize adhesion, gently press or rub from center out to edges of tape, 3) To promote skin health, do not stretch or apply with tension over areas where tape will be adhering directly to skin, 4) Refer to "3M Trouble-free Taping" brochure for additional usage recommendations.

For more information contact your 3M Health Care Sales Representative or call the 3M Health Care Customer at **1-800-228-3957**. These products can be ordered from your local distributor. Outside the United States, contact the local 3M subsidiary.



**Skin & Wound Care Division**  
**3M Health Care**  
 3M Center, Building 275-4W-02  
 St. Paul, MN 55144-1000  
 USA  
 1-800-228-3957  
[www.3M.com/healthcare](http://www.3M.com/healthcare)

**3M Canada Company**  
 P.O. Box 5757  
 London, Ontario N6A 4T1  
 Canada  
 1-800-364-3577  
[www.3M.com/ca/healthcare](http://www.3M.com/ca/healthcare)

3M and Micropore  
 are trademarks of 3M.  
 Please recycle. Printed in U.S.A.  
 © 3M 2000, 2010. All rights reserved.  
 70-2010-8114-1