



RAPID ONE-STEP SCREENING TEST

SURESTEP™ URINE TEST E-Z SPLIT KEY® CUP

- Integrated test cup screens for 6-8 prescription and illicit drugs
- Begin testing at your convenience with unique key activator
- Fast test result in 5 minutes
- Self-contained collection cup and test to minimize exposure to urine
- Drug cutoff levels as per AS/NZS 4308:2008




Device image may vary based on test configurations.

Integrated E-Z Split Key® Cup Urine Drug Screen


ASSAY	CALIBRATOR	*CUTOFF LEVEL (ng/mL)
Amphetamine (AMP 300)	d-Amphetamine	300
Benzodiazepines (BZO 200)	Oxazepam	200
Buprenorphine (BUP)	Buprenorphine	10
Cocaine (COC 300)	Benzoylcegonine	300
Marijuana (THC)	11-nor- Δ^9 -THC-9 COOH	50
Methadone Metabolite (EDDP 300)	EDDP	300
Methamphetamine (MET 300)	d-Methamphetamine	300
Opiates (MOP 300)	Morphine	300

*Drug cutoff levels as per AS/NZS 4308:2008


SIMPLE COLLECTION AND TESTING PROCEDURE

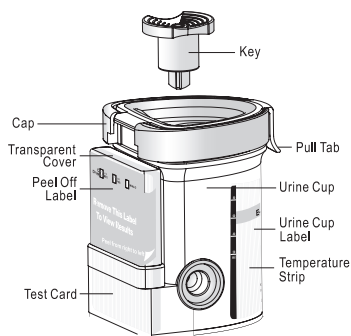


1 Collect specimen and snap test lid shut.

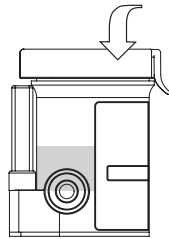

2 Collector uses key to plunge specimen and activate test.

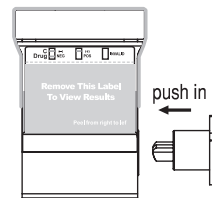

3 Peel label to reveal test and specimen validity (if applicable). Read test results at 5 minutes.

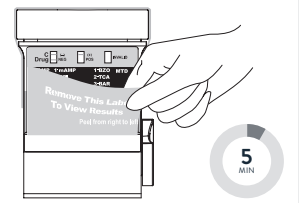

PRODUCT PROCEDURE NOTE: Refer to product insert for complete instructions, limitations, and warnings. This assay provides only a preliminary analytical test result. A more specific alternative chemical method must be used in order to obtain a confirmed analytical result. Gas chromatography-mass spectrometry (GC-MS) is the preferred confirmatory method. Clinical consideration and professional judgment should be applied to any drug test result, particularly when preliminary positive results are indicated.

FOR MORE INFORMATION:

EMAIL: RAPIDDX.ANZ.ENQUIRIES@ABBOTT.COM
 PHONE: AUSTRALIA: 1800-622-642
 NEW ZEALAND: 0800-466-951